



Medisafe - PCD Kit Datasheet


The PCD (Process Challenge Device) and CEI (Cleaning Efficacy Indicator) kit has been designed to closely replicate a hard-to-clean dirty instrument, and indicate the effectiveness of your ultrasonic cleaner.

Supplied in the Kit -

- Stainless Steel tray to hold CEI slide during test
- Multiple test CEI test slides

How to use -

- Place the CEI slide inside the stainless steel tray
- Making sure the CEI is secure and the tray is closed place inside the ultrasonic cleaner you wish to test.
- Allow the cleaning cycle to complete and remove the tray.
- Carefully remove the CEI slide and check that the soiled area has been cleaned sufficiently.
- Please see the chart below for more information.

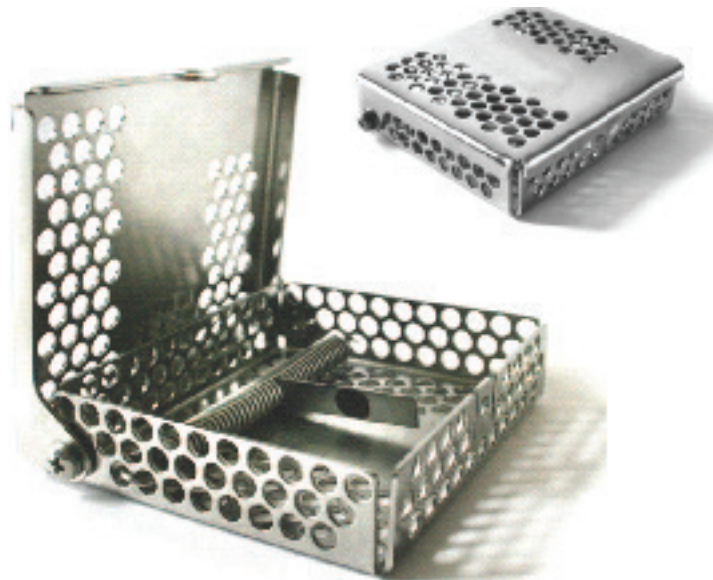

More information -

Name	Medisafe PCD Kit
Classification	ISO 15883/ HTM01-05
Reusable / single use	Reuseable PCD tray
Non-toxic	Yes

Results	Assessment	Action Needed
	Reference indicator: Spot of test soil completely visible	
Insufficient cleaning process:		
	Spot of test soil completely visible	Ultrasonic Cleaner Failed
	Spot of test soil visible	Check wash, temperature, dosage and time.
	Spot of test soil visible	Repeat test.
	Spot of test soil visible	Repeat test.
Successful cleaning process:		
	Spot of test soil completely removed	Test Successful. No further action needed